

# Delta Tips

NMDT\_0051

## How to Set Default Data Folder

NMR data processing software

**Delta**  
NMR Software  
v5.0




Delta software can be customized in many ways. In this issue of Delta Tips, we will demonstrate how the default data folder can be changed permanently by editing Preferences. We will change the default data folder from 'Delta data' to 'test data'.



'test data' is a new folder on the Desktop



- ① Select **Options — Preferences** in the **Delta** window to open **Preferences** window.

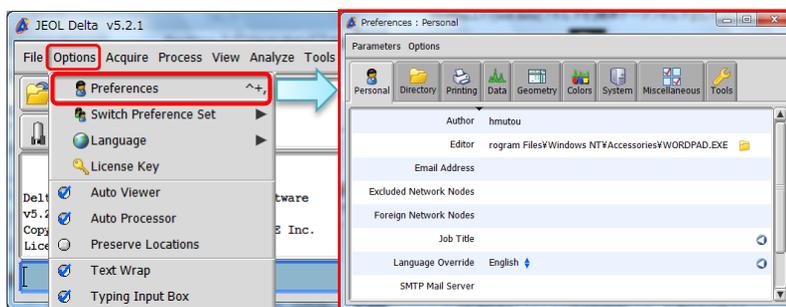

- ② Click the **Directory** button to display the **Directory** tab.

- ③ Click the **Directory** button  on the right side of the **Data** item in the **Directory** tab.

Note that the **Directory Tool** window has opened.

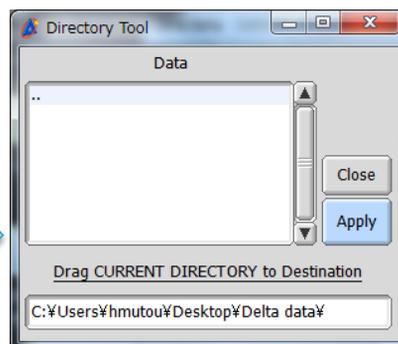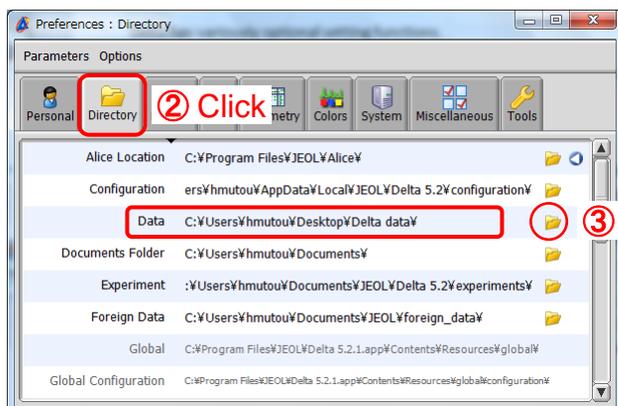

**Directory Tool** window

### Directory tab


The **Data** parameter defines the location of the default data folder

- ④ Push and hold the right-mouse-button on 'test data' folder shortcut




Note that the cursor has changed to the **Select symbol**


- ⑤ Drag and drop it onto the input box in the **Directory Tool** window.

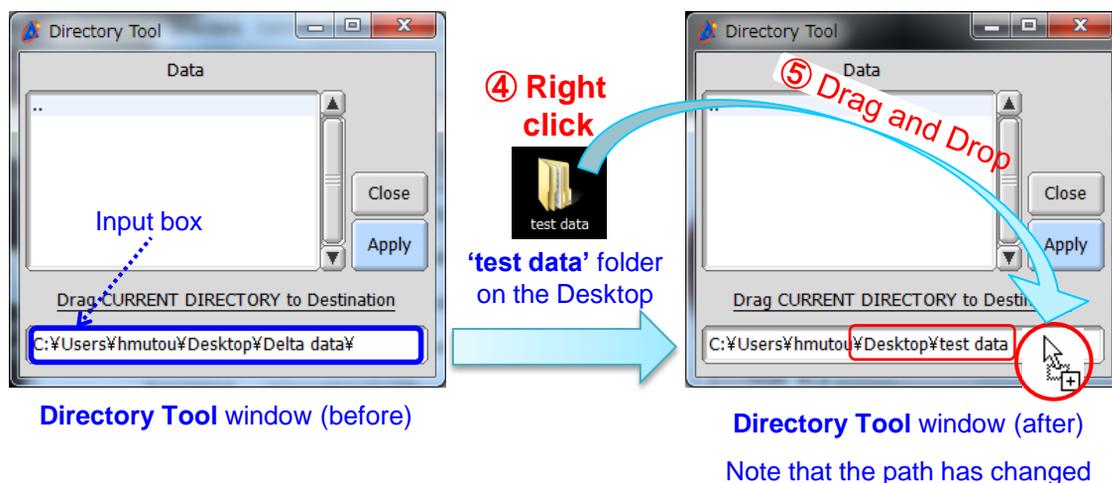

- ⑥ Push the **Apply** button , close the **Directory Tool** window and the **Preferences** window. Click the **OK** button in the **Inform** window which has opened.

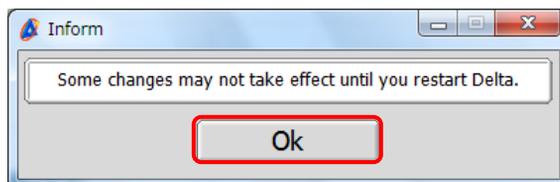

- ⑦ Restart **Delta** to apply the new setting.

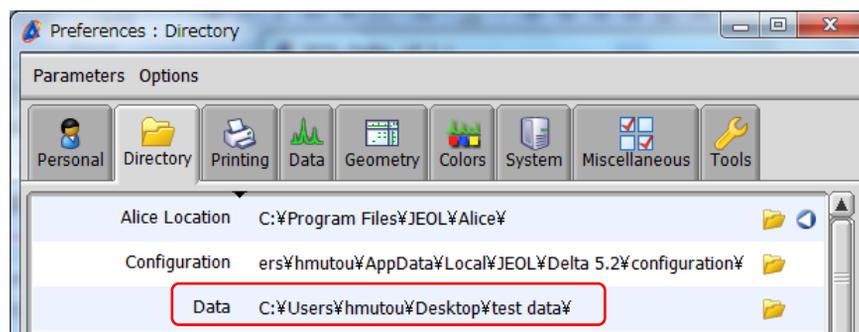

The default data folder has changed to 'test data'